

REACH TECHNOLOGY

MULTI-VOICE FOCUS

Multi-Voice Focus* scans the surrounding environment 1000 times per second with four independent focus beams. This enables it to constantly adapt to the changing dynamics of the conversation and make the experience fluid and comfortable. It will boost the prominence of those in the group who are speaking while keeping track of those who are silent, and all without switching between modes or programs, and with no delay.

VOICE STABILIZER

Voice Stabilizer controls the compression to secure that all voices within the focus beam are amplified in the best linear way when the sound gets too loud. It keeps the signal-to-noise ratio positive to help users focusing on the signal with most contrast.

MY VOICE 2.0

General sound and the users own voice are processed separately, but in parallel. In this way we can ensure that the impression of the users own voice stays comfortable while not interfering with the ambient signal processing.

CONNECTIVITY

Reach hearing aids can stream phone calls and audio from iPhone via MFi and Android via ASHA and LE Audio**. When connected via MFi or LE Audio, users can enjoy hands free calls via Headset Mode. With this feature there is no need to hold the phone in your hand for a call. The hearing aids microphones will pick up the users voice so that the user can be on a call without needing to stop what they are doing.

While Headset Mode is active, this will enable Call Control and Tap Control. The user can now accept, reject or terminate a phone call via the hearing aid.

LIFEPROOF

Rexton hearing aids are built using robust components and are tested extensively to ensure they can withstand whatever life throws at them. You can rely on them to withstand everyday challenges and harsh environments.

ATEX

The majority of Reach form factors has ATEX*** certification which verifies that they are tested and approved to be intrinsically safe for use in potentially explosive environments.

* Only available for hearing aids with two microphones in a binaural fitting.

** Please make sure to confirm the compatibility to the users specific phone since the functionality depends on the phone manufacturers enablement.

*** They can be safely used in areas where there may be flammable substances in the air such as paint and chemical manufacturing, petrol stations and mills. The ATEX group/class is printed on the ATEX card, which must be available at hand when the hearing aids are used in explosive atmospheres.

REACH R-LI AND R-LI T




- › Enabling users to be fully involved in group conversations with Multi-Voice Focus
- › Handsfree phone calls via Headset Mode with Call Control when connected via MFi or LE Audio
- › Portable charger with built-in power bank
- › Rechargeable solutions for a full days use:
 - › R-Li delivers 24h runtime (incl. 5h streaming)
 - › R-Li T delivers 34 h runtime (incl. 5h streaming)
- › Lifeproof to withstand everyday challenges
- › ATEX certified
- › AI with Rexton Assist helps you to seamlessly customize your hearing aids

Colors



Fitting range is based on 2 ccm coupler:

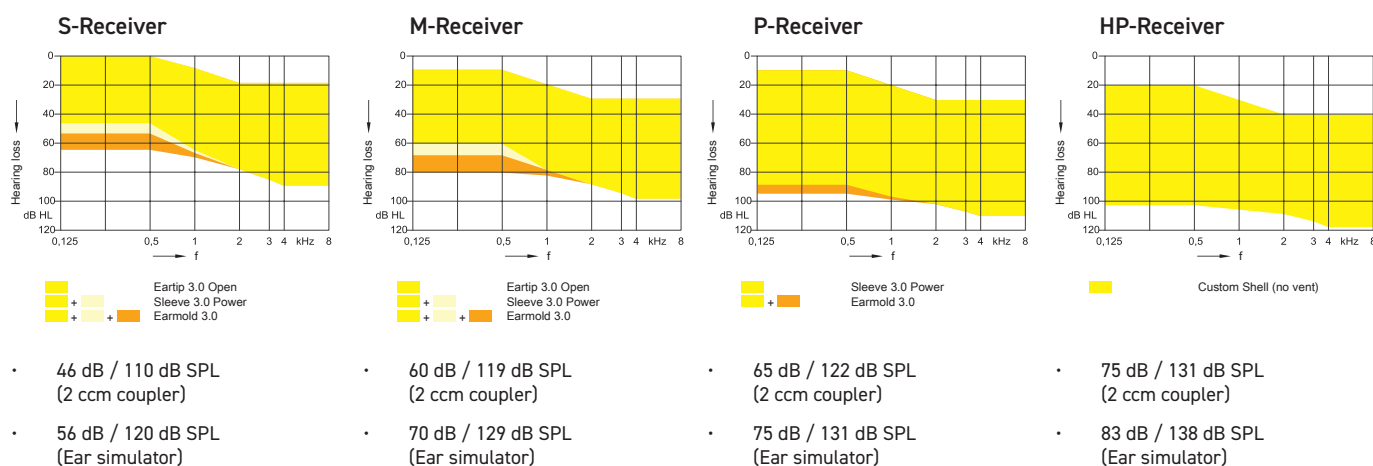

Features / Performance Level		80	60	40	30	20
Channels/Controls/Programs		48/20/6	36/18/6	32/16/6	24/12/4	16/8/4
T-coil		R-Li T	R-Li T	R-Li T	R-Li T	R-Li T
Bluetooth		•	•	•	•	•
Tinnitus Sound function		•	•	•	•	-
Tinnitus Notch function		•	•	•	•	-
Wireless sync		•	•	•	•	•
IP68		•	•	•	•	•
XPhone		•	•	•	•	•
Technology Highlights						
Multi-Voice Focus		•	•	•	-	-
Focus Beams		4	2	1	1	1
Voice Stabilizer		•	•	•	•	•
Speech Preservation Technology		•	•	•	•	•
My Voice 2.0		•	•	•	-	-
Motion sensor for scene analysis		•	•	•	-	-
Directionality						
Automatic adaptive		•	•	•	•	•
iOmni Sound Locator		•	•	•	•	•
Auto iFocus 360		•	-	-	-	-
iFocus 360		•	•	-	-	-
Auto Focus 360		-	-	-	-	-
Auto Stereo iLock		•	•	•	-	-
OneMic Directionality		-	-	-	-	-
Noise Reduction						
Noise Management		•	•	•	•	•
Sound Smoothing		•	•	•	•	•
Wind Noise Reduction		•	•	•	•	•
Auto Echo Reducer		•	-	-	-	-
Reverb Reducer		•	•	-	-	-
Sound Quality & Control						
Bandwidth extension		•	-	-	-	-
Bandwidth compression		•	•	•	•	•
Feedback preventer		•	•	•	•	•
Music Enhancer (presets)		3	3	1	1	-
Automatic Acclimatization		•	•	•	•	•
Audio Streaming						
LE Audio		•	•	•	•	•
MFi		•	•	•	•	•
ASHA		•	•	•	•	•
Headset Mode						
LE Audio		•	•	•	•	•
MFi		•	•	•	•	•
TeleCare						
Full live remote tuning		•	•	•	•	•
Basic portal remote tuning		•	•	•	•	•
Accessories & Apps		Chargers		CROS		
Smart Key	•	Charging Station	•	Reach CROS R-Li	•	
Smart Transmitter 2,4	•	Travel Charger	•	Reach CROS SR	-	
Smart Mic	•	Charging+ Station	•	Reach CROS iX-CIC Li	-	
Rexton App	•	IF CIC Travel Charger	-			
		Universal Cable Charger	•			
Programming						
Noahlink Wireless	•					
ConnexxAir	-					
HI-PRO	-					

REACH SR



- › Enabling users to be fully involved in group conversations with Multi-Voice Focus
- › Handsfree phone calls via Headset Mode with Tap Control when connected via MFi or LE Audio
- › Portable charger with built-in power bank
- › Rechargeable solution for a full days use:
 - › SR delivers 17h runtime (incl. 5h streaming)
- › Lifeproof to withstand everyday challenges
- › ATEX certified
- › AI with Rexton Assist helps you to seamlessly customize your hearing aids

Colors

Black | Black Gloss



Cosmic Blue | Rose Gold



Snow White | Rose Gold



Snow White | Snow White Gloss




Black | Silver



Fitting range is based on 2 ccm coupler:


S-Receiver

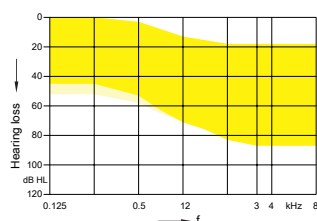

EarTip 3.0 Open
 Sleeve 3.0 Power

- 46 dB / 110 dB SPL (2 ccm coupler)
- 56 dB / 120 dB SPL (Ear simulator)


M-Receiver

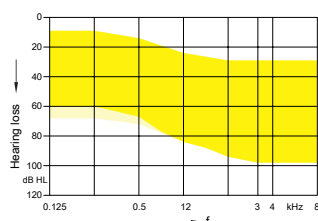

EarTip 3.0 Open
 Sleeve 3.0 Power

- 60 dB / 119 dB SPL (2 ccm coupler)
- 70 dB / 129 dB SPL (Ear simulator)

P-Receiver

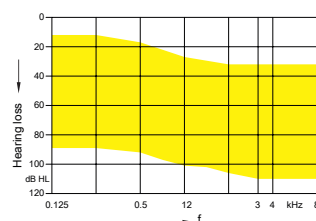

Sleeve 3.0 Power

- 65 dB / 122 dB SPL (2 ccm coupler)
- 75 dB / 131 dB SPL (Ear simulator)

Features / Performance Level		80	60	40	30	20
Channels/Controls/Programs		48/20/6	36/18/6	32/16/6	24/12/4	16/8/4
T-coil		-	-	-	-	-
Bluetooth		•	•	•	•	•
Tinnitus Sound function		•	•	•	•	-
Tinnitus Notch function		•	•	•	•	-
Wireless sync		•	•	•	•	•
IP68		•	•	•	•	•
XPhone		•	•	•	•	•
Technology Highlights						
Multi-Voice Focus		•	•	•	-	-
Focus Beams		4	2	1	1	1
Voice Stabilizer		•	•	•	•	•
Speech Preservation Technology		•	•	•	•	•
My Voice 2.0		•	•	•	-	-
Motion sensor for scene analysis		•	•	•	-	-
Directionality						
Automatic adaptive		•	•	•	•	•
iOmni Sound Locator		•	•	•	•	•
Auto iFocus 360		•	-	-	-	-
iFocus 360		•	•	-	-	-
Auto Focus 360		-	•	-	-	-
Auto Stereo iLock		•	•	•	-	-
OneMic Directionality		-	-	-	-	-
Noise Reduction						
Noise Management		•	•	•	•	•
Sound Smoothing		•	•	•	•	•
Wind Noise Reduction		•	•	•	•	•
Auto Echo Reducer		•	-	-	-	-
Reverb Reducer		•	•	-	-	-
Sound Quality & Control						
Bandwidth extension		•	-	-	-	-
Bandwidth compression		•	•	•	•	•
Feedback preventer		•	•	•	•	•
Music Enhancer (presets)		3	3	1	1	-
Automatic Acclimatization		•	•	•	•	•
Audio Streaming						
LE Audio		•	•	•	•	•
MFi		•	•	•	•	•
ASHA		•	•	•	•	•
Headset Mode						
LE Audio		•	•	•	•	•
MFi		•	•	•	•	•
TeleCare						
Full live remote tuning		•	•	•	•	•
Basic portal remote tuning		•	•	•	•	•
Accessories & Apps						
Smart Key	•					
Smart Transmitter 2,4	•					
Smart Mic	•					
Rexton App	•					
Programming						
Noahlink Wireless	•					
ConnexxAir	-					
HI-PRO	-					
Chargers						
Charging Station	-					
Travel Charger	•					
Charging+ Station	•					
IF CIC Travel Charger	-					
Universal Cable Charger	-					
CROS						
Reach CROS R-Li	-					
Reach CROS SR	•					
Reach CROS iX-CIC Li	-					

REACH INOX-CIC LI

THE FIRST INVISIBLE HEARING AID WITH RECHARGEABILITY




- > With Voice Stabilizer the contrast in every conversation is controlled for easier speech intelligibility
- > OneMic Directionality enables the benefits of a directional microphone
- > A discreet design with easy handling makes it virtually invisible
- > Portable charger with built-in power bank
 - > inoX-CIC Li delivers 28h runtime
- > Lifeproof to withstand everyday challenges
- > ATEX certified

Faceplate colors



Fitting range is based on 2 ccm coupler:

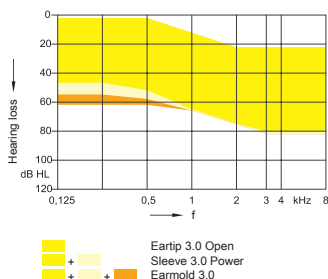

- 50 dB / 114 dB SPL
(2 ccm coupler)
- 61 dB / 126 dB SPL
(Ear simulator)



For a customized, yet invisible rechargeable fit, Reach inoX-CIC Li is compatible to earmold fittings.

Features / Performance Level		80	60	40	30	20
Channels/Controls/Programs		48/20/6	36/18/6	32/16/6	24/12/4	16/8/4
T-coil		-	-	-	-	-
Bluetooth		-	-	-	-	-
Tinnitus Sound function		•	•	•	•	-
Tinnitus Notch function		•	•	•	•	-
Wireless sync		•	•	•	•	•
IP68		•	•	•	•	•
XPhone		•	•	•	•	•
Technology Highlights						
Multi-Voice Focus		-	-	-	-	-
Focus Beams		-	-	-	-	-
Voice Stabilizer		•	•	•	•	•
Speech Preservation Technology		-	-	-	-	-
My Voice 2.0		-	-	-	-	-
Motion sensor for scene analysis		-	-	-	-	-
Directionality						
Automatic adaptive		•	•	•	•	•
iOmni Sound Locator		-	-	-	-	-
Auto iFocus 360		-	-	-	-	-
iFocus 360		-	-	-	-	-
Auto Focus 360		-	-	-	-	-
Auto Stereo iLock		-	-	-	-	-
OneMic Directionality		•	•	•	-	-
Noise Reduction						
Noise Management		•	•	•	•	•
Sound Smoothing		•	•	•	•	•
Wind Noise Reduction		•	•	•	•	•
Auto Echo Reducer		•	-	-	-	-
Reverb Reducer		•	•	-	-	-
Sound Quality & Control						
Bandwidth extension		•	-	-	-	-
Bandwidth compression		•	•	•	•	•
Feedback preventer		•	•	•	•	•
Music Enhancer (presets)		3	3	1	1	-
Automatic Acclimatization		•	•	•	•	•
Audio Streaming						
LE Audio		-	-	-	-	-
MFi		-	-	-	-	-
ASHA		-	-	-	-	-
Headset Mode						
LE Audio		-	-	-	-	-
MFi		-	-	-	-	-
TeleCare						
Full live remote tuning		-	-	-	-	-
Basic portal remote tuning		•	•	•	•	•
Accessories & Apps		Chargers		CROS		
Smart Key	•	Charging Station		Reach CROS R-Li		-
Smart Transmitter 2,4	-	Travel Charger		Reach CROS SR		-
Smart Mic	-	Charging+ Station		Reach CROS iX-CIC Li		•
Rexton App	•	IF CIC Travel Charger				
		Universal Cable Charger				
Programming						
Noahlink Wireless	-					
ConnexxAir	•					
HI-PRO	-					

REACH CUSTOM IIC AND CIC

A NEXT-LEVEL CUSTOM MADE INVISIBLE



- > With Voice Stabilizer the contrast in every conversation is controlled for easier speech intelligibility
- > OneMic Directionality enables the benefits of a directional microphone
- > Our smallest customized solution yet
- > Battery driven solutions with 36% better battery lifetime compared to predecessor:
 - > Custom IIC and CIC (battery size 10) delivers up to 75h runtime
- > Lifeproof to withstand everyday challenges

Colors

Black




Brown



Mocha



Fitting range is based on 2 ccm coupler:

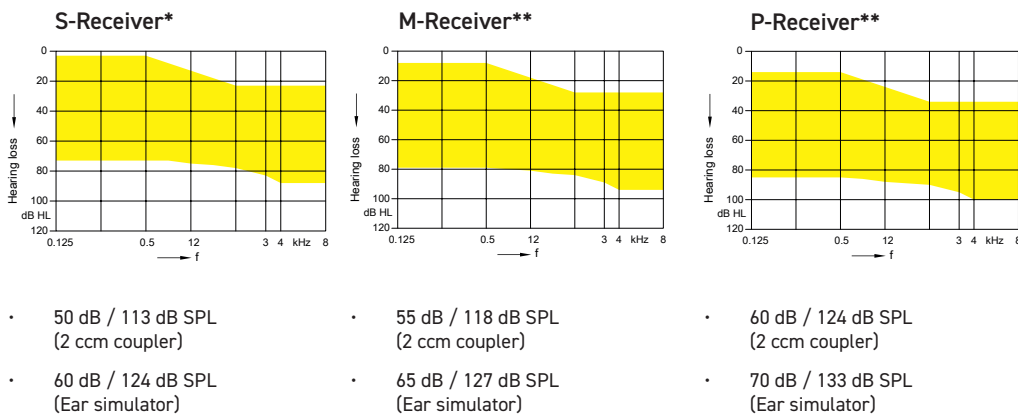

* Available for both IIC and CIC

** Available for CIC

Features / Performance Level		80	60	40	30	20
Channels/Controls/Programs		48/20/6	36/18/6	32/16/6	24/12/4	16/8/4
T-coil		-	-	-	-	-
Bluetooth		-	-	-	-	-
Tinnitus Sound function		•	•	•	•	-
Tinnitus Notch function		•	•	•	•	-
Wireless sync		•	•	•	•	•
IP68		-	-	-	-	-
XPhone		-	-	-	-	-
Technology Highlights						
Multi-Voice Focus		-	-	-	-	-
Focus Beams		-	-	-	-	-
Voice Stabilizer		•	•	•	•	•
Speech Preservation Technology		-	-	-	-	-
My Voice 2.0		-	-	-	-	-
Motion sensor for scene analysis		-	-	-	-	-
Directionality						
Automatic adaptive		•	•	•	•	•
iOmni Sound Locator		-	-	-	-	-
Auto iFocus 360		-	-	-	-	-
iFocus 360		-	-	-	-	-
Auto Focus 360		-	-	-	-	-
Auto Stereo iLock		-	-	-	-	-
OneMic Directionality		•	•	•	-	-
Noise Reduction						
Noise Management		•	•	•	•	•
Sound Smoothing		•	•	•	•	•
Wind Noise Reduction		•	•	•	•	•
Auto Echo Reducer		•	-	-	-	-
Reverb Reducer		•	•	-	-	-
Sound Quality & Control						
Bandwidth extension		•	-	-	-	-
Bandwidth compression		•	•	•	•	•
Feedback preventer		•	•	•	•	•
Music Enhancer (presets)		3	3	1	1	-
Automatic Acclimatization		•	•	•	•	•
Audio Streaming						
LE Audio		-	-	-	-	-
MFi		-	-	-	-	-
ASHA		-	-	-	-	-
Headset Mode						
LE Audio		-	-	-	-	-
MFi		-	-	-	-	-
TeleCare						
Full live remote tuning		-	-	-	-	-
Basic portal remote tuning		•	•	•	•	•
Accessories & Apps						
Smart Key		•				
Smart Transmitter 2,4		-				
Smart Mic		-				
Rexton App		•				
CROS						
Reach CROS R-Li			-			
Reach CROS SR			-			
Reach CROS iX-CIC Li			•			
Programming						
Noahlink Wireless		-				
ConnexxAir		•				
HI-PRO		•				